

Healthweigh® Indicators

Adjusting the Power Down Time Limit

Product: Healthweigh Indicator

Issue: By default, the indicator automatically powers down after 9 minutes when powered with batteries only

Solution: Adjust the TOFF setting

When the healthweigh indicator is powered by batteries only, it powers down automatically after 9 minutes. Power down time can be configured from 1-9 minutes or disabled.



Manuals and additional resources are available from Rice Lake Weighing Systems at www.ricelake.com/manuals

Warranty information is available at www.ricelake.com/warranties

Use the following procedure to adjust the display time out setting:

1. Remove the four screws securing the back cover to the indicator (see [Figure 1](#)).

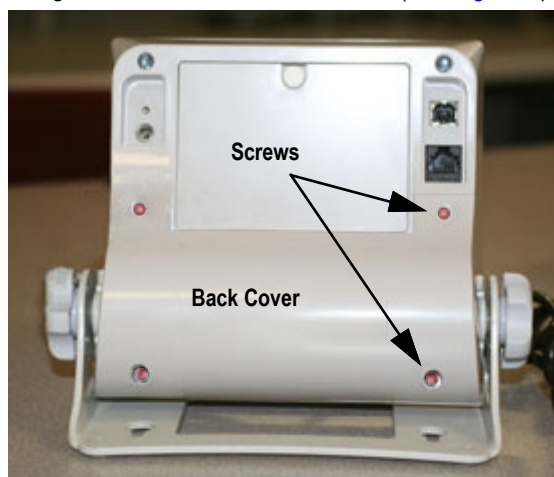


Figure 1. Back Cover Screw Locations

2. Press and hold the **ON/OFF** and **ENTER** keys until the ID displays.

3. Press the setup switch on the back of the indicator just to the right of load cell cable (see [Figure 2](#)). PROG displays.

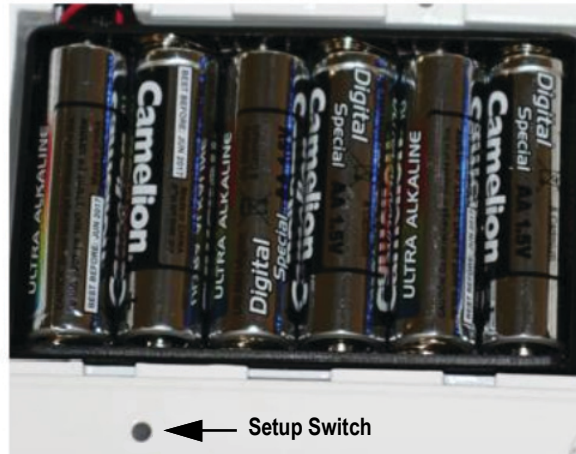


Figure 2. Setup Switch Location

4. Press **ENTER**.
5. Press the **BMI** key until TOFF displays.
6. Press **ENTER**. The blinking number is the number of minutes the scale will wait to power down when operating with batteries.
7. Use the down arrow key to change the minute value from 1-9, as needed.



IMPORTANT: If value is set to 0, then it stays powered on until the ON/OFF key is pressed to power down the scale.



© Rice Lake Weighing Systems Content subject to change without notice.

230 W. Coleman St. • Rice Lake, WI 54868 • USA USA: 800-472-6703 • International: +1-715-234-9171