## Rice Lake Adult & Child Scale Model D400 Calibration Instructions

The Rice Lake Adult and Child Scale takes the traditional home health scale to a new level, by accurately weighing a child while the child is being held. The scale is powered by battery or AC adapter. The scale should always be placed on a hard flat surface, avoiding carpeted areas.



This manual can be viewed and downloaded from the Rice Lake Weighing Systems web site at <u>www.ricelake.com</u>

Use the following steps to calibrate the scale.

- 1. Place the power switch under scale in the *OFF* position.
- 2. Toggle the KG/LB switch to put the scale into kilograms.

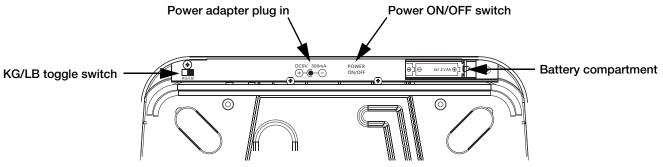


Figure 1. Bottom of the Scale

- 3. Press and hold the On/Off button on the top of the scale, while moving the power switch to the ON position. The top button must be held until power is turned on.
- 4. Press and release the On/Off button on the top of the scale four times continuously.
- 5. Immediately press the button again and hold it for 3-4 seconds. The raw count of the scale displays.(Example *301000*)
- 6. Press the On/Off button on the top of the scale. 50 kg displays.
- 7. Place 50 kg on the center of the scale. Within a few seconds 100 kg displays.
- 8. Place 100 kg on the center of the scale. Within a few seconds 150 kg displays.
- 9. Place 150 kg on the center of the scale. PASS displays indicating the calibration was accepted.
- 10. Power the scale off and then back on.



If the weights are not accepted and the calibration locks up, power cycle the unit and recalibrate. If the calibrate calibration fails again, the scale may be defective or need servicing. Contact Rice Lake Weighing Systems.



© Rice Lake Weighing Systems Specifications subject to change without notice. Rice Lake Weighing Systems is an ISO 9001 registered company.

230 W. Coleman St. • Rice Lake, WI 54868 • USA U.S. 800-472-6703 • Canada/Mexico 800-321-6703 • International 715-234-9171 • Europe +31 (0)26 472 1319

www.ricelake.com



Figure 2. On/Off Button